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Mobile Stand „AssistStand“

TABLE OF CONTENTS

	Page
1. Introduction	12
2. Intended Use / Range of Application / Contraindication	12
3. Safety Information	13
4. Scope of delivery / Accessories / Specifications	14
5. Assembly Instructions	15
6. Cleaning	17
7. Storage	17
8. Disposal	17
9. Warranty / Durability	17
10. Maintenance	18
11. Reuse / Safety Control	18
12. EC Declaration of Conformity	19

AssistStand (without accessories)

Instruction Manual - Mobile Stand „AssistStand“

1. INTRODUCTION

Dear Customers,

many thanks for buying the AssistStand! You have purchased a mobile workstand of high quality, which has been developed and tested according to recent EU standards. We are sure that the AssistStand in conjunction with an adequate computer hard- and software will make a valuable contribution to the integration of handicapped people.

The individual adjustment options and simple handling will be a great help and a facilitation for the patients and the nursing staff.

2. INTENDED USE / RANGE OF APPLICATION / CONTRAINDICATION

The AssistStand has been designed as a monitor stand and workstation in the field of private care for patients. It is used as a support and positioning aid for PCs, communication aids and monitors.

The AssistStand allows patients with impaired communication or motor skills for the basic need for communication or PC control.

This innovative, ergonomic and easy-to-maneuver healthcare cart features infinitely variable height adjustment and also has a screen positioning regardless of standing, lying or sitting postures.

Through optional accessories such as a keyboard tray or PC platform the AssistStand will be an ideal mobile workstation.

The drivability of the AssistStand is an important precondition for a fast installation of the hard- and software inside the field of vision of the patient. For the purpose of caring the patient the AssistStand can easily and safely be pulled over.

It can accommodate all commercially available monitors up to a maximum weight of 6 kg, if they have 100 mm on a mounting option VESA standard 75 x 75 mm to 100 x 100 mm.


Contraindicated are patients who move uncontrollable their arms and legs and therefore endanger the stability of the AssistStand.

3. SAFETY INFORMATION



- Please read the instruction manual completely before use and mounting. Here you get important details for a safe use of your AssistStand. The manufacturer does not assume liability for failures and damages which result from non-observance of this instruction manual.
- Through purchasing and commercial use of this medical device you assume the **USER DUTIES**. Please read up upon these duties.
- Do not fit any monitor to the AssistStand, whose weight is more than the maximum weight of 6 kg.
- Do not fit any other equipment or units to the AssistStand. If you mount any units which are not approved by the manufacturer any warranty claim and the product liability in general will expire. The same applies if units belonging to the AssistStand are left out.
- Move the AssistStand only if the monitor is pushed up against the pole. Move particularly slowly and cautious across structural hindrances like steps and edges of carpets. Otherwise there is a risk of tilting!
- Moving is only allowed when the brakes of the casters are released. Move the AssistStand only by catching hold on the rear handle.
- Lock the casters only if the AssistStand has been come to halt.
- In the course of mounting and moving the monitor arm there is a risk of injury in the form of crushing skin and fingers at moving parts.

Instruction Manual - Mobile Stand „AssistStand“





4. SCOPE OF DELIVERY / ACCESSORIES / SPECIFICATIONS

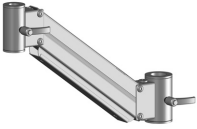Units	Description	Weight	Illustration
1	Base with Running Casters (pre-assembled)	19,74 kg	
1	Pole 1.025 mm	5,54 kg	
1	Rear Handle with Clamp Screw	0,62 kg	
1	Pole Mount Bracket with Set Screw and Allen Key 6 mm	0,15 kg	
1	Monitor Arm with Gas Spring, Locking Screw and Set Screw	2,58 kg	
1	VESA Display Support with Fastening Module incl. Fastening Washer and Screw M8 x 45 <i>Note on Installation:</i> Before assembling remove fastening washer and screw M8 x 45.	0,80 kg	

ACCESSORIES (optionally available)

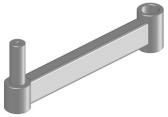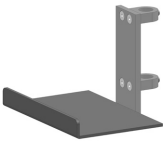Units	Description	Weight	Illustration
1	Extension Arm for the Monitor Arm (300 mm)	2,66 kg	
1	CPU Holder with Pole Mount Bracket <i>Note on Installation:</i> Fit the CPU holder and fasten it with the pole mount brackets between step 1 and 2.	2,40 kg	
1	Pole Extension (400 mm) <i>Note on Installation:</i> Unfasten the black plastic end cap from the 1.025 mm pole and bolt the pole extension to the 1.025 mm pole hand-screwed. The assembly steps 2 to 6 apply to the pole extension correspondingly.	2,60 kg	


5. ASSEMBLY INSTRUCTIONS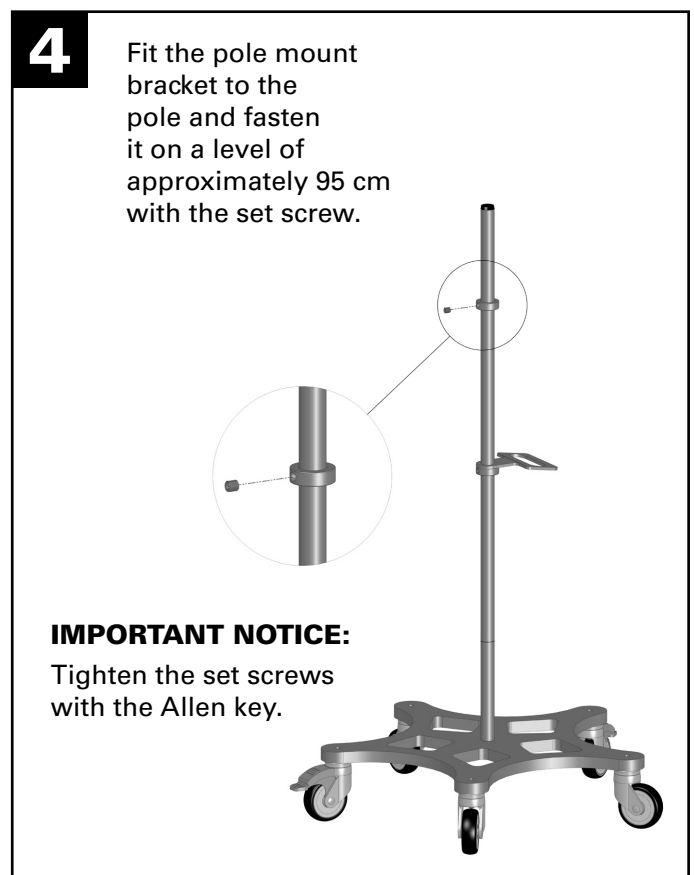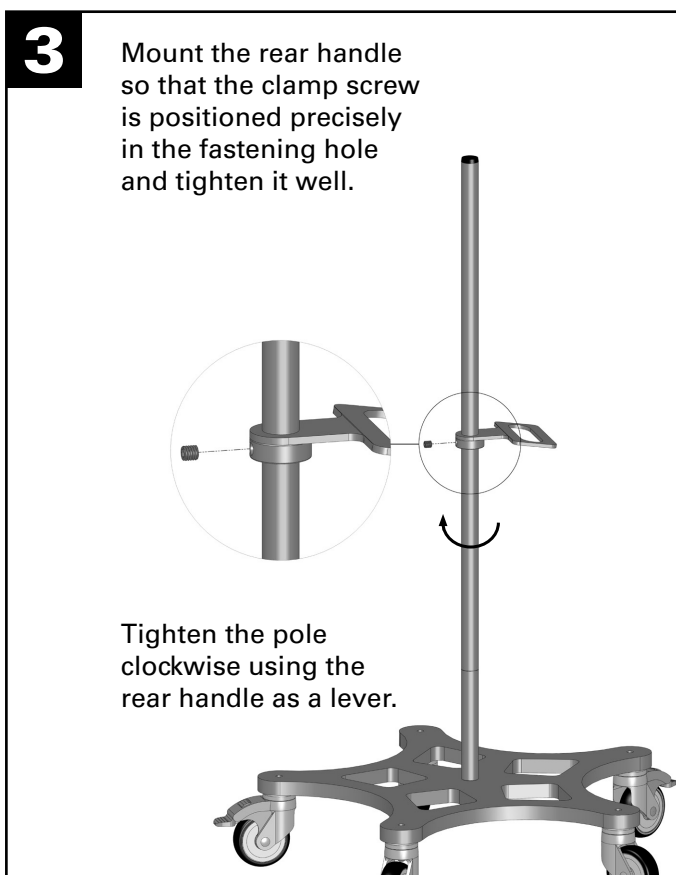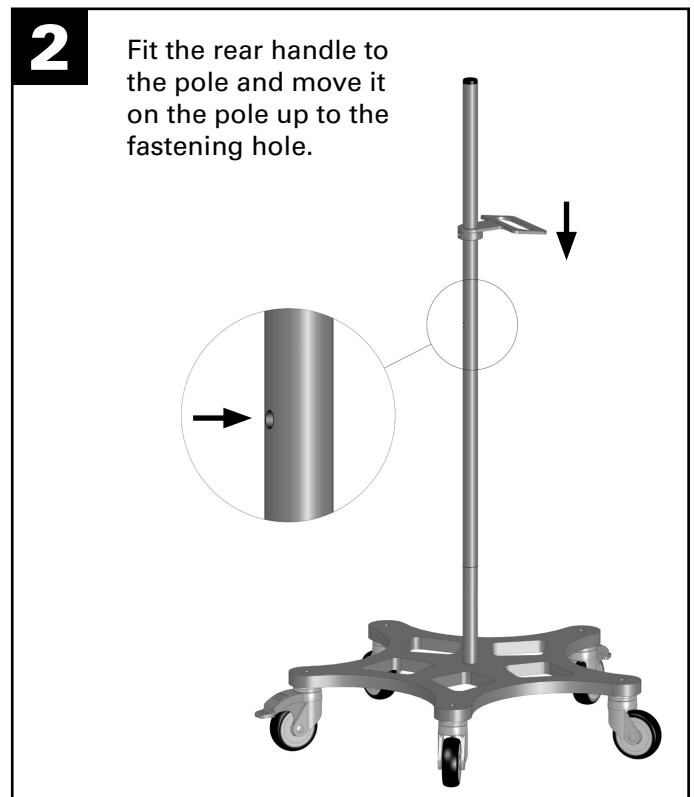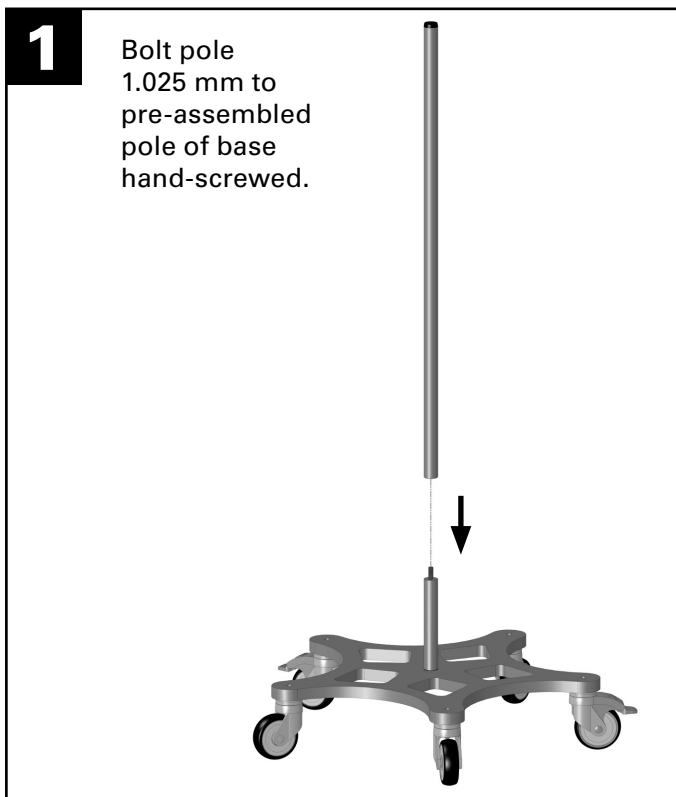

5


Mounting of the Extension Arm (Accessory):

Push the extension arm on the pole up against the pole mount bracket.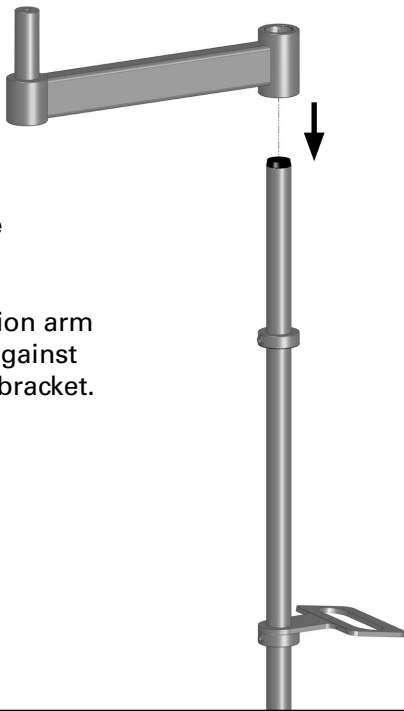


6

Push the monitor arm on the pole up against the pole mount bracket. If necessary, the arm can be fixed with the locking screw.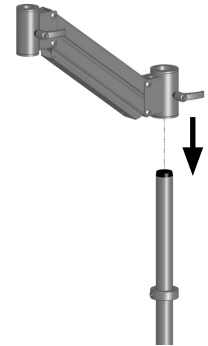

With Extension Arm (Accessory):

Fit the monitor arm to the extension arm and lock it with the fastening washer. If necessary, the arm can be fixed with the locking screw.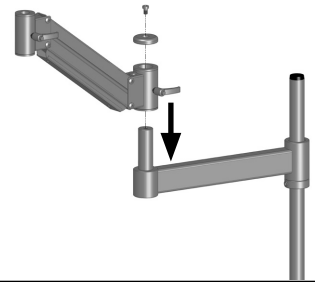


7

The VESA Display support is inserted from the bottom into the lift arm and...


A) secured with the set screw

B) can be locked with the locking screw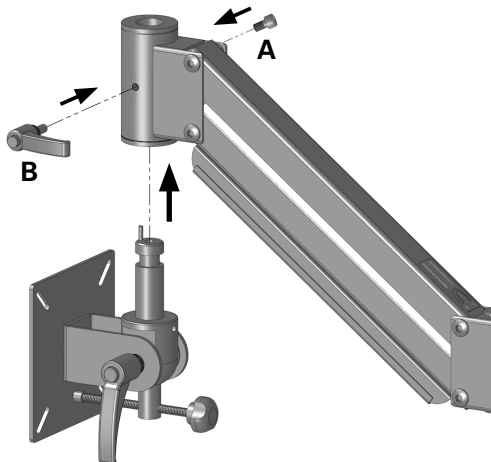

8

Fit the fastening washer from above and tie it together with the VESA Display Support by using screw M8 x 45.

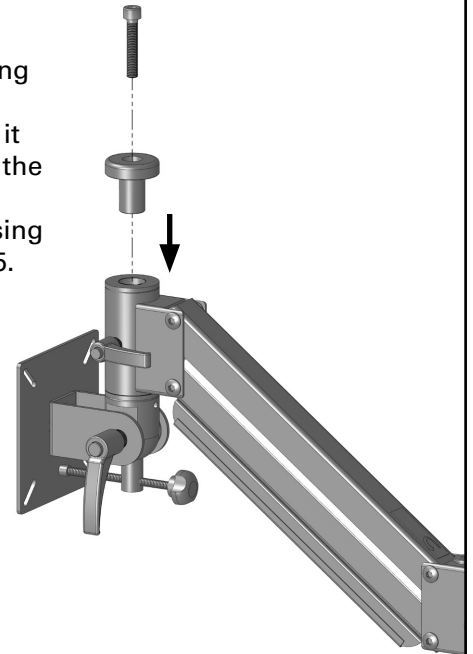6. CLEANING

Clean the AssistStand with a wet cloth and an usual pH-neutral household cleaner. Do not use aggressive substances like ammonium chloride, cleaning powder or thinner.

Use for wiping disinfection mild and gentle disinfectants which are registered by the DGHM (Deutsche Gesellschaft für Hygiene und Mikrobiologie).

7. STORAGE

The AssistStand is adapted for multiple further use. For storing disassemble the monitor stand in reversed order compared to the assembly instructions (Chapter 5) and pack it dry and dust free. Due to improper storage in a wet environment rust can be formed at the column.

Pay attention to the safety instructions in Chapter 11 before reuse.

Clean and disinfect the AssistStand before every further use and carry out an maintenance according to chapter 10. Use only mild and gentle solutions for disinfection and cleaning, according to the disinfectant list of the VAH (Verbund für Angewandte Hygiene).

If the AssistStand is transported to a reuse and the original packaging is no longer available, you must ensure yourself that components are not damaged by vibrations or other harmful influences.

8. DISPOSAL

For disposal disassemble the AssistStand and bring the components to a scrap metal collection place. Do not dispose it by the domestic waste. Unpressure the gas spring of the monitor arm and depollute the oil. For this purpose contact the manufacturer of the AssistStand.

9. WARRANTY / DURABILITY

We guarantee the unobjectionable condition of the AssistStand within the limits of our general terms and conditions. We guarantee material defects for 24 months from date of purchase.

It is assumed that the product will have a durability of more than five years when its handled carefully. Should the AssistStand still be in use after ten years then it should be sent in to the manufacturer for safety check.

Instruction Manual - Mobile Stand „AssistStand“

10. MAINTENANCE

Put the AssistStand at least once a year and before every further use to a maintenance comprehending the following articles:

- Check the running casters up if they are tightened and free-moving.
- Lock the brackets of the two lockable running casters and check the braking effect.
- Retighten all screws with the 6 mm Allen key. These are the set screws of the pole mount brackets, the monitor arm, the VESA display support and the rear handle.
- Check by bouncing the pole if it is fastened carefully at the base. Retighten it if necessary. If using the optional 400 mm pole extension also check if it is fastened carefully and retighten it if necessary.
- Do use only mild and gentle solutions for cleaning and disinfection, according to the disinfectant list of the VAH (Verbund für Angewandte Hygiene).
- Check the function and prestressing of the gas spring in the monitor arm by moving the monitor up and down. It should easily be movable upwards and not slide down. If the monitor slides down self-acting the gas spring has to be adjusted. If it is not possible to adjust the gas spring it has to be changed.

In case of dysfunctions inform the operator who will contact the manufacturer of the AssistStand. Here the spare parts can be quickly ordered. **Only qualified and skilled personnel is allowed to replace the components.**

The gas spring is a closed system which is irreparable. You must **NOT OPEN** it in any case!

11. REUSE / SAFETY CONTROL

If the AssistStand is transported to a reuse and the original packaging is no longer available, you must ensure yourself that components are not damaged by vibrations or other harmful influences.

12. EC DECLARATION OF CONFORMITY

Manufacturer	EDS Warenhandelsgesellschaft für ergonomische Arbeitsmittel mbH Antwerpener Straße 6 53842 Troisdorf Germany
Product	AssistStand
Classification according to Article 9 (1) of the EC Council Directive 93/42/EEC (MDD)	Class 1 Rules 1 according to Appendix IX
Conformity assessment procedure according to Article 11 (5) of the EC Council Directive 93/42/EEC	Declaration of conformity according to Appendix VII

We hereby declare in sole responsibility that the medical products mentioned above comply with the requirements of the EC Council Directive 93/42/EEC (Medical Products) and EC Council Directive 2007/47/EC. The associated documentation is kept in the premises of the manufacturer.

Applicable standards	ISO 15375:2004 -in dependence- EN 12182:1999-11 EN 62366:2008 EN 60601-1-6:2004
-----------------------------	--

Place, Date	Troisdorf, 15 July 2016
--------------------	-------------------------

Signature	 Ernst Jäger Managing Director
------------------	---

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Hersteller/Producer: EDS GmbH